SSL-LHD-30

Wireless LED Headlight Camera System

 Sunoptic Technologies®
6018 Bowdendale Avenue
Jacksonville, FL 32216 USA

Customer Service: 904 737-7611
Toll Free: 877 677-2832

EC|REP AJW Technology Consulting GmbH
Breite Straße 3
40213 Düsseldorf, Germany
Telephone: [+49 211 54059 6030](tel:+49211540596030)

TABLE OF CONTENTS

- 1. INTRODUCTION**
 - 1.1. Indications for Use
 - 1.2. Contraindications
 - 1.3. Function of Design
 - 1.4. Classifications
 - 1.5. Package Contents
- 2. WARNINGS AND SYMBOLS**
 - 2.1. Warnings and Cautions
 - 2.2. Symbols
- 3. PRODUCT OVERVIEW**
 - 3.1. Recording Unit Buttons and Indication Lights
- 4. INSTALLATION**
 - 4.1. Battery Installation
 - 4.2. SD Card Installation
 - 4.3. HD Camera Attachment
 - 4.4. HD Video Receiver Setup
- 5. SYSTEM STARTUP AND OPERATION**
 - 5.1. Wireless Transmission Verification
 - 5.2. Camera Focus
 - 5.3. Using the Camera
 - 5.4. File Handling
- 6. SYSTEM SETTINGS**
 - 6.1. Settings Navigation
 - 6.2. System Language
 - 6.3. System Date/Time
 - 6.4. Format SD Card
 - 6.5. Factory Reset
- 7. BATTERIES AND CHARGING**
 - 7.1. Battery and Charger Warnings
 - 7.2. Low Battery Signa
 - 7.3. Battery Charger Specifications
 - 7.4. Battery Charger Operation
- 8. CLEANING AND DISINFECTION**
- 9. MAINTENANCE, SERVICING, WARRANTY AND REPAIR**
 - 9.1. User Service and Maintenance
 - 9.2. Limited Warranty
 - 9.3. Repair
- 10. TROUBLESHOOTING**
 - 10.1. System Malfunction
 - 10.2. Interrupted Wireless Connection
 - 10.3. File Save Failure
 - 10.4. Image Blur
 - 10.5. Artifacts in Image
- 11. END OF PRODUCT LIFE**

1. INTRODUCTION

Thank you for the purchase of a new Wireless LED Headlight Camera System!

The Wireless LED Headlight Camera System is an LED headlight-mounted surgical camera used to capture still and video images of surgical procedures.

You have chosen the best and we would like to make sure you receive optimal results with your new camera system through correct operation.

1.1. Indications for Use

The SSL-LHD-30 is intended to display and record surgical procedures. This device is not intended to diagnose, prevent, monitor, treat or alleviate disease, injuries or handicaps, nor for the investigation, replacement or modification of the anatomy or of a physiological process, and is not to be used with or as an endoscope or related system.

The SSL-LHD-30 has no essential performance as defined by IEC 60601-1. Users are advised that should this system fail to operate, have suitable backup systems available for any procedure in which this device may be used.

1.2. Contraindications

There are no known contraindications associated with this device.

1.3. Function of Design

The Wireless LED Headlight Camera System comprises of a Recording Unit, HD Camera, Wireless HD Video Receiver, and the base.

- The Main Recording unit is powered by two rechargeable lithium-ion batteries and can be attached to the operator via rear clip.
- The HD camera can be attached to a headlight through various connectors depicted later in this manual and is also connected to the main recording unit.
- The HD video receiver is connected to a monitor.
- Once communication between the main recording unit and the wireless HD video receiver has been established, images and video captured by the HD camera can be displayed on a remote monitor.

1.4. Classifications

Requirements	Standard
Radio Spectrum	ETSI EN 301 893 V1.6.1
EMC	ETSI EN 301 489-1 V1.8.1 ETSI EN 301 489-17 V2.1.1 FCC 47 CFR Part 15 E
Safety	EN 60950-1:2006 + A11:2009 + A1:2010
Health	EN 62311:2008
General	CE Directive 1999/5/EC, RoHS 2011/65/EU

1.5. Specifications

Parameter	Value
Operating Environment	5°C – 40°C (41°F - 104°F) 20 – 80% R.H., non-condensing 86 – 106kPa
Storage Environment	-10°C – 55°C (14°F - 131°F) 10 – 95% R.H., non-condensing 86 – 106kPa
Video file	1080p @ 30fps, MP4
Image file	1080p, JPEG
Working distance	Adjustable 20 – 60 cm (≈ 8 – 23.6 in)
Depth of field	20 cm (≈ 8 in)
Control method	Buttons on Recording Unit or remote-control fob
microSDXC card capacity	128 GB
Battery Duration	3 hours
Wireless transmission	1080p up to 20m (≈ 65 ft)
HD Video Receiver power input	100 – 240V ~ 50/60Hz
Recording Unit weight	270g (≈ 9.5oz)
HD Camera weight	45g (≈ 1.6oz) excluding cable
Water resistance	Not protected equipment, IPX0

1.6. Package Contents

ID	Description	Qty
1	Recording Unit	1
2	HD Camera + attachment	1
3	HD Video Receiver + Power Adaptor	1
4	Rechargeable Lithium-Ion Batteries + Cases	4
5	microSDXC Card	1
6	Battery Charger + Power Cable	1
7	Remote Control Fob + Lanyard	1
8	Micro-USB Cable	1
9	USB Type C Cable	1
10	HD Video Receiver Base	1
11	6ft HDMI Cable	1
12	Locking Carrying Case	1

2. WARNINGS AND SYMBOLS

2.1. Warnings and Cautions

Use of this equipment may present hazards to the user and/or patient. Before operating this device, please read this operating manual thoroughly and follow all warnings, cautions, and instructions for use. The words warning, caution, and note carry special meaning and should be carefully reviewed:

WARNING: Indicates risks to the safety of the patient or user. Failure to follow warnings may result in injury to the patient or user.

CAUTION: Indicates risks of improper use and/or damage to the equipment. Failure to follow cautions may result in loss of function or product damage.

NOTE: Indicates special information to clarify instructions or present additional useful information.

The appropriate “WARNING”, “CAUTION” or “NOTE” symbol in this manual is intended to alert the user to the presence of important operating and maintenance instructions in the manual.

WARNING:

- Failure to follow instructions in this manual may lead to serious injury or damage to the equipment. Read this operating manual thoroughly, especially the warnings, and be familiar with its contents before using this equipment.
- This equipment is designed to be used by a qualified professional, having complete knowledge of the use of this equipment and the procedure to be performed.
- This equipment should be installed and tested prior to use. A pre-operative check should be performed prior to administration of patient anesthesia to ensure all desired functions are operational, that a viable surgical image is being displayed on the monitor, and there are no signs of damage to the equipment.
- It is recommended to have a back-up system ready for use in the event of primary equipment failure.
- Use only manufacturer’s specified compatible accessories and peripheral equipment. Use of non-approved equipment may cause loss of function and void of warranty.
- Before each use, check the outer surface of this equipment to be used to ensure that there are no rough surfaces, sharp edges, or protrusions that can cause injury.
- Avoid dropping the camera system or rough handling. The camera system contains sensitive components prone to damage by mechanical shock.
- Do not use the equipment in the presence of flammable liquids, gases, or other materials susceptible to ignition due to electrical sparking.
- This equipment generates and radiates RF energy which may affect the normal function of nearby installed equipment. Conversely, equipment that generates and radiates RF energy may affect the normal function of this device. Perform a compatibility check by running both devices in proximity before use in a medical procedure.
- Carefully unpack this unit and check if any damage occurred during shipment. If damage is detected, refer to the Warranty and Return Policy section of this manual.
- There is a risk of damage to the equipment if the equipment is subjected to cleaning or sterilization methods not approved by the manufacturer. To reduce risk of loss of function

and/or damage to the equipment, only use the approved cleaning and sterilization methods described herein.

- Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State and/or Country in which the user and/or patient is established

NOTE: The warranty may become void if any of these warnings or cautions are disregarded.

2.2 Symbols

Symbol	Description
	Manufacturer
	Date of manufacture (YYYY-MM-DD)
	Symbol for "Authorized Representative in the European Community."
	Caution, consult accompanying documents
	Consult Instructions for Use
	CE mark
	Not for disposal in general waste
	AC Current
	Power on
	Power off
	Do not use if package is damaged
	Medical device
	Non-Sterile
	Storage / Shipping Temperature
	Storage / Shipping Humidity
	Barometric Pressure
	Unique Device Identifier
	Keep Dry

3. PRODUCT OVERVIEW

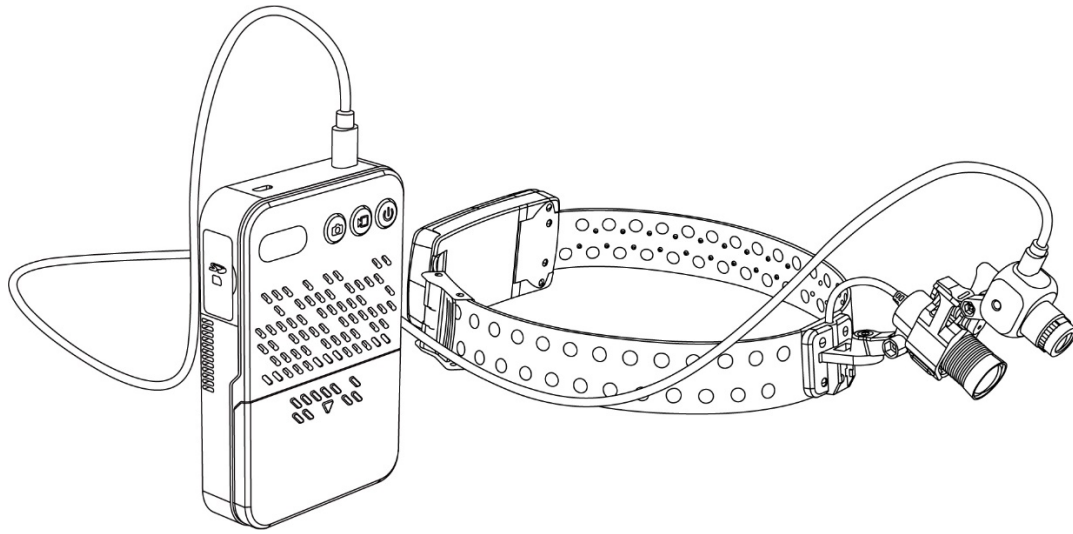
Figure 1 (above). HD Camera attached to generic headlight and Recording Unit

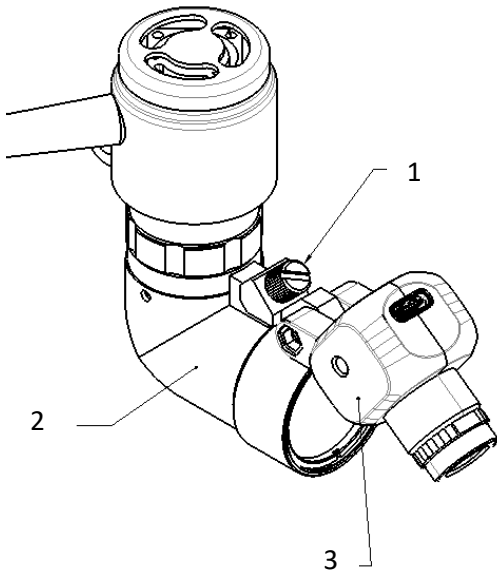
Figure 2 (left). HD Camera attached to Sunoptic Surgical LED Headlight

1. Thumbscrew
2. LED Headlight
3. HD Camera

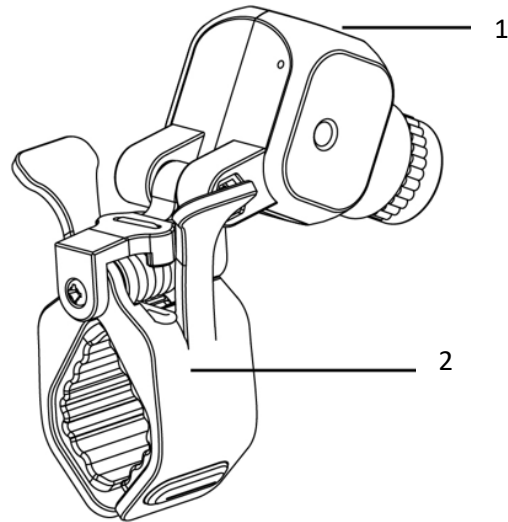
Figure 3 (right). HD Camera with optional clamp mount

1. HD Camera
2. Clamp mount

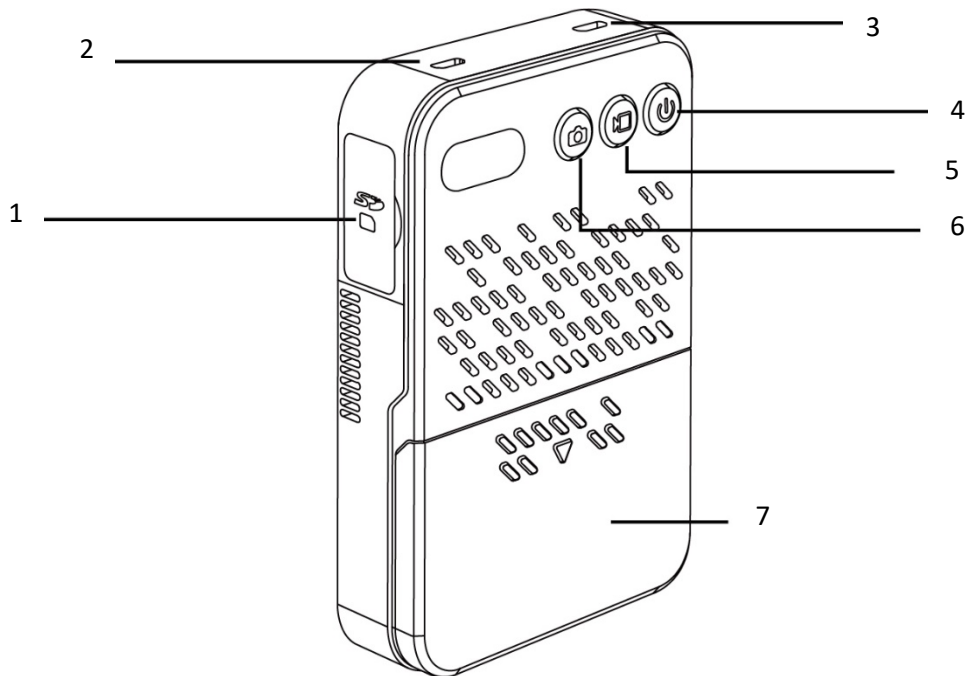
Figure 4. Recording Unit

- | | |
|----------------------------|------------------------|
| 1. SD Card Slot | 5. Recording Button |
| 2. Micro-USB Connection | 6. Capture Button |
| 3. USB-C Camera Connection | 7. Battery Compartment |
| 4. ON/OFF Button | |

3.1. Recording Unit Buttons and Indication Lights

The Recording Unit is equipped with three buttons as depicted in *Figure 4*. Each button can act as a control input and/or status indicator. Please reference Tables 1.

Icon	Name	Function
	ON/OFF Button	<ul style="list-style-type: none"> Press and hold to start up or shut down Illuminates blue when system is ON.
	Record Button	<ul style="list-style-type: none"> Short press to initiate or terminate video recording Flashes blue when recording No illumination when not recording
	Capture Button	<ul style="list-style-type: none"> Short press to capture still image Flashes blue light once to indicate captured image.

Table 1. Recording Unit Button Control

4. INSTALLATION

4.1. Battery Installation

NOTE: Batteries are provided in a discharged state. It is recommended to fully charge all batteries before using the device.

WARNING: Inspect batteries prior to use. Do not use any battery that has any visible signs of damage or deformation.

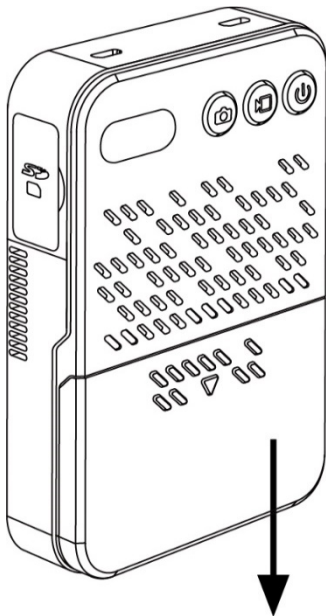
Figure 5 (left). Slide battery compartment cover down and fully install two of the provided batteries into the slots, noting the polarity inscribed inside of the compartment. Replace the battery compartment cover by sliding upward until it clicks.

4.2. SD Card Installation

NOTE: The Recording Unit must be powered off when installing the microSDXC Card, otherwise the Recording Unit will automatically power down.

Open the SD Card rubber cover and insert the microSDXC Card with the contacts facing toward the front of the Recording Unit into the slot. Press down until the card clicks into place. Replace the rubber cover.

After a new SD Card is installed, the card will need to be formatted. Refer to **Format SD Card** for detailed instructions.

4.3. HD Camera Attachment

Install the HD Camera on a Sunoptic Surgical LED Headlight using the thumbscrew provided as depicted in *Figure 2*. or attach the HD Camera onto a generic headlight using the clamp depicted in *Figure 3*.

NOTE: The HD camera has the potential to fall if not securely mounted to a headlight. Please ensure that the HD Camera is securely fastened to a headlight prior to use.

Connect the HD Camera to the Recording Unit with the provided USB-C cable. The Recording Unit can be connected to an HD Camera in either the ON or OFF state.

NOTE: When attaching or detaching the USB-C cable, grab the cable by the connector housing instead of the wire to avoid damage to the cable.

4.4. HD Video Receiver Setup

Connect the provided power adaptor to the HD Video Receiver. Connect the provided HDMI cable from the back of the HD Video Receiver to an appropriate monitor.

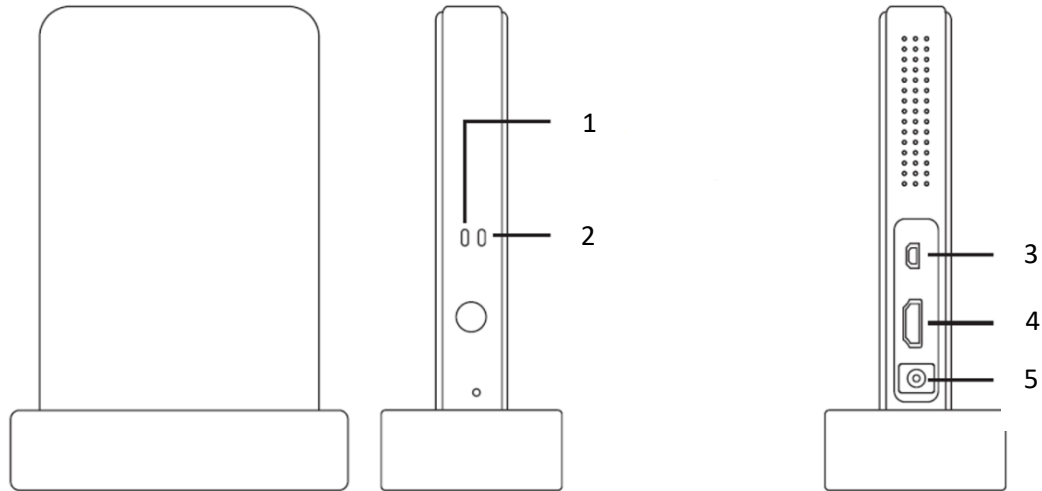
Figure 6. HD Video Receiver

- | | |
|----------------------------|---------------------|
| 1. Network Indicator Light | 4. HDMI Output |
| 2. Video Indicator Light | 5. Input Power Jack |
| 3. Mini-USB (not used) | |

5. SYSTEM STARTUP AND OPERATION

To turn on the Recording Unit, Press and hold the ON/OFF button for three to five seconds until the device beeps.

5.1. Wireless Transmission Verification

Check the Network and Video Indicator lights. An illuminated Network Indicator light signifies that communication has been established between the Recording Unit and HD Video Receiver. An illuminated Video Indicator light shows that the HD Video Receiver is receiving and transmitting a video signal. If either indicator light is off, then restart the Recording Unit and/or reposition the HD Video Receiver.

After the Recording Unit and HD Receiver have initialized, the camera image will be displayed on the screen.

5.2. Camera Focus

Camera focus may be adjusted by rotating the lens-holding ring located along the camera shaft until the image becomes clear. The focus can be locked into place by tightening the gear-shaped ring against the HD Camera body.

5.3. Connecting External Monitor

- Recording video
 - Short press the Record Button on the Recording Unit. The device will beep, the Record Button will flash blue, and a red dot will flash at the top, right corner of the display.

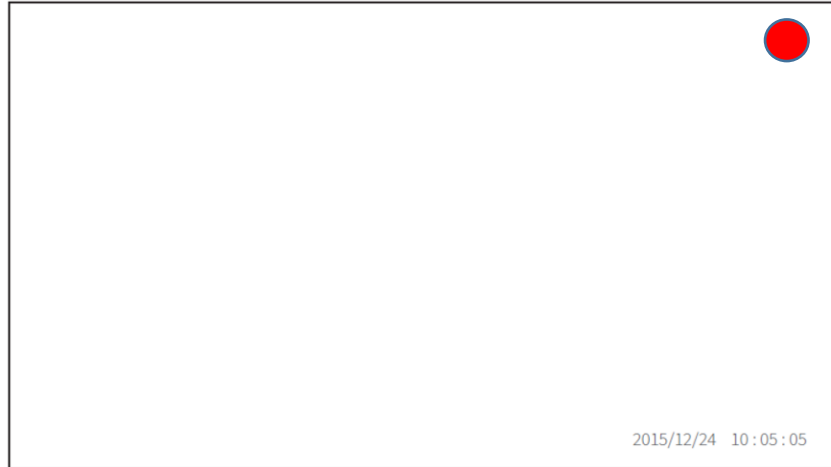
Figure 7. On-Screen Display During Recording

- Capture still image
 - Still images can be captured at any time after system start up by pressing the Capture Button

5.4. Connecting Mains Power

- Recorded video is saved as .MP4 format in a “DCIM\Movie” directory that is automatically created on the microSDXC card.
- Captured images are saved as .JPG format in a “DCIM\Photo” directory that is also automatically created on the microSDXC card.
- A 128GB microSDXC card can store up to 20 hours of video.
- The system is configured to store video recordings in a fixed time interval. If a video is shorter than this interval, the video will be saved as one file. If a video is longer than this interval, the recording file will be segmented and stored as multiple files.
- Video and Image files will be saved according to the following naming format:
 - DATE + TIME + SERIAL NUMBER or YYYY_MMDD_HHMMSS_XXX
- When the microSDXC card runs out of available space, previous files will be overwritten automatically. Please ensure to backup files regularly.
- While turned ON, the Recording Unit file system can be browsed by a connected PC through the Micro-USB connector.

NOTE: Browsing the Recording Unit file system is only possible when the unit is not recording.

6. SYSTEM SETTINGS

After initializing the Recording Unit and HD Video Receiver, press and hold the Capture Button to enter the System Settings. Below is a list of all Menu items:

- Exposure
- Metering
- Saturation
- Frequency
- Record Audio
- Language
- Date Stamp
- Date/Time
- Format
- Default Setting
- Version

NOTE: The camera system color composition and saturation has been calibrated according to factory configuration. Users should only change these settings if necessary.

6.1. Settings Navigation

Icon	Action	Description
	Move Up or Left	Short press the Capture Button to move up or left in the menu system
	Move Down or Right	Short press the Record Button to move down or right in the menu system
	Enter Sub Menu	Long press the Capture Button to enter the sub menu of the selected menu item
	Cancel	Long press the Record Button to discard changes and exit System Settings
	Confirm	Long press the Capture Button to save changes

Table 2. System Settings Menu Navigation Controls

6.2. System Language

English	English	Francais	French
Español	Spanish	Português	Portuguese
Deutsch	Germany	Italiano	Italian
简体中文	Simplified Chinese	繁體中文	Traditional Chinese
Русский	Russian	日本語	Japanese

Table 3. Available Languages

- Long press the Capture Button to enter System Settings.
- Repeatedly press the Record Button to navigate to the Language section of System Settings.
- Long press the Capture Button to highlight the language select.
- Once selected, press either the Record or Capture buttons to cycle to the desired language.
- To confirm the highlighted language, long press the Capture Button.

6.3. System Date/Time

- Long press the Capture Button to enter System Settings.
- Repeatedly press the Record Button to navigate to the Date/Time section of System Settings.
- Long press the Capture Button to enter the Date/Time sub menu.
- Use the Record and Capture buttons to navigate to the value.
- Long press the Capture Button to confirm selection and to advance to the next section.

6.4. Format SD Card

- Long press the Capture Button to enter System Settings.
- Repeatedly press the Record Button to navigate to the Format section of System Settings.
- Long press the Capture Button to initiate Format Dialog Box.
- Press the Record Button to select OK.
- Long press the Capture Button to confirm selection

6.5. Factory Reset

- Long press the Capture Button to enter System Settings.
- Repeatedly press the Record Button to navigate to the Default Setting section of System Settings.
- Long press the Capture Button to initiate Return to Default Dialog Box.
- Press the Record Button to select OK.
- Long press the Capture Button to confirm selection

7. BATTERIES AND CHARGING

7.1. Battery and Charger Warnings

- Upon initial receipt and before each use, inspect each battery for damage. Do not attempt to charge or use a damaged battery. Damaged batteries must be correctly disposed, immediately.
- Do not expose a battery to heat, fire, mechanical shock, or store in direct sunlight.
- Do not open or dismantle a battery.
- Do not sterilize, immerse, or expose a battery to water, solvents, lubricants, or other chemicals. Do not allow water to collect in the battery contacts. Keep clean and dry.
- Do not short-circuit, crush, or shred a battery.
- Do not store the batteries haphazardly in a box or drawer in which the batteries may short circuit each other or be short-circuited by other metal objects. It is recommended to store unused batteries in the provided plastic containers.
- In the event of a battery leakage, do not allow the liquid to contact skin or eyes. If contact has been made, wash the affected area with copious amounts of water and seek medical advice.
- Do not use any charger other than the charger supplied by the manufacturer.
- Ensure correct battery orientation is followed before placing into a device or charger.
- Do not attempt to force a battery into a device.
- When possible, remove the battery from equipment when not in use.
- Do not over-discharge the battery. Ensure that a battery is promptly charged in an appropriate charger.
- Do not store batteries in a discharged state.
- Do not store batteries for more than one year without recharge.
- Do not leave charger unattended while connected to power.
- Do not attempt to charge a non-rechargeable battery.
- Operate charger in a well-ventilated area.

NOTE: The batteries and/or charger may become warm during the charging process.

7.2. Low Battery Signal

The Recording Unit will beep when the battery charge becomes low.

7.3. Battery Charger Specifications

Input: 100-240VAC 50/60Hz 0.35A(MAX), 12VDC 1A

Output Voltage: 4.2VDC \pm 1% / 3.7VDC \pm 1% / 1.48VDC \pm 1%

Output Current: 375mA x 4 / 750mA x 2

Dimension: 143mm x 99mm x 36mm

Weight: 239g (without batteries and power cord)

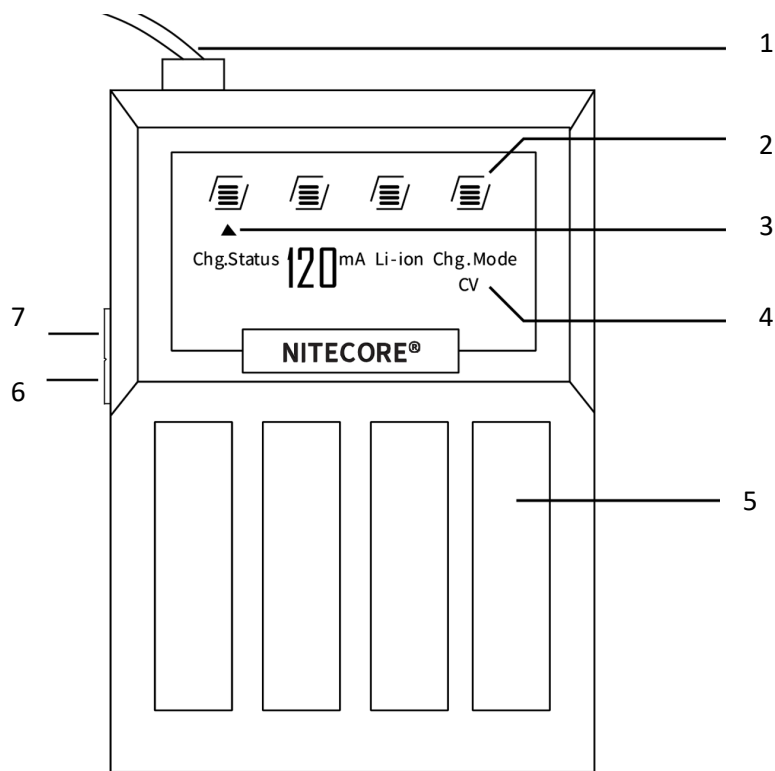
Figure 8. Battery Charger

- | | |
|------------------------------------|-----------------|
| 1. AC Power Cord | 5. Battery Bays |
| 2. Battery Indicator Bars | 6. MODE Button |
| 3. Currently Selected Battery Icon | 7. SLOT Button |
| 4. Battery Charge Information | |

7.4. Battery Charger Operation

Inspect the charger for signs of damage before use. Connect the charger to a power source and install batteries into the battery bays, noting proper orientation.

When batteries are first installed into the charger, the charger performs a battery health check causing the battery indicator bars to illuminate in sequence from bottom to top. When the test is complete, the charger will enter charging mode. If any unusual characteristics are detected during the test, such as a battery installed backwards, the battery indicator bars will rapidly flash, and the screen will display an “Err” message. Users should remove the battery and install again. If the “Err” message persists, the battery should be disposed.

While in charging mode, the Battery Indicator Bars will blink. Once a battery is fully charged, all Battery Indicator Bars will be fully illuminated. The charger will show battery and charging information of the currently selected battery slot. Users can cycle through this information by pressing the MODE button. To show information on another slots, press the SLOT button. To turn the LCD backlight off, press, and hold the SLOT button.

NOTE: Holding the MODE button will cycle the charger charge setting between LiFePO4 or Li-ion chemistry cells. Always have the charger chemistry setting to “Li-ion” when charging the batteries provided with the camera system.

8. CLEANING AND DISINFECTION

The Wireless LED Headlight Camera System may be cleaned, but not sterilized. **Remove batteries and/or unplug from power source before cleaning.**

CAUTION: Never immerse or sterilize any part of the camera system as this will damage the device and void the warranty.

The surfaces of the HD Camera, Recording Unit, and HD Video Receiver can be wiped down with commercially available cleansers commonly used for disinfection of electronic equipment in hospitals such as ethyl or isopropyl alcohols, disinfecting sprays containing quaternary ammonium compounds, or hydrogen peroxide. The lenses should be cleaned only with lens tissue.

CAUTION: Do not use strongly caustic or acidic cleansers such as “Clorox” hypochlorite bleach, ammonia, muriatic acid, or similar products. Do not use acetone, methyl ethyl ketone, halogenated or chlorinated hydrocarbon solvents or cleansers containing any of these restricted compounds.

Ensure that the SD Card slot cover is in place before cleaning the Recording Unit. Apply cleaning agents by light spray to a cloth or towel. Do not spray directly on equipment.

CAUTION: Do not pour liquids onto the device. Do not allow frank liquids to enter either the camera head or camera console seams or ventilation openings.

Follow all applicable blood borne pathogen procedures as required by OSHA and/or your hospital when cleaning and disinfecting the product

9. MAINTENANCE, SERVICING, WARRANTY AND REPAIR

9.1. User Service and Maintenance

There are no user serviceable parts on the camera head nor the camera console. Defective items or equipment are to be serviced and repaired exclusively by persons authorized by the manufacturer. All repair work shall employ original manufacturer parts only.

9.2. Limited Warranty

The Wireless LED Headlight Camera System possesses a 2-year warranty from the date of shipment on workmanship and all defects of material excluding the rechargeable batteries, which will carry a 1-year warranty. Should your product prove to have such defects within two years of shipment, **Sunoptic Surgical®** will repair or replace the product or component part without charge. Should your product(s) need servicing under this warranty, please contact **Sunoptic Surgical®** for return authorization documentation. Please contact **Sunoptic Surgical®** for packing instructions. Include a note describing the defects, your name, telephone number and a return address. Warranty does not cover equipment subject to misuse, accidental damage, normal wear, and tear or if transferred to a new owner without authorization from **Sunoptic Surgical®**. This warranty gives you specific legal rights and you may also have other rights that vary from state to state.

9.3. Repair

The Wireless LED Headlight Camera System may be returned to the factory with prepaid shipping. The device will be inspected, and an estimate of repair charges will be submitted for approval.

PHONE: +1 (877) 677-2832 INTERNATIONALS: +1 (904) 737-7611
FAX: +1 (904) 733-4832

10. TROUBLESHOOTING

10.1 System Malfunction

Symptom: The status indicator lights on the HD Video Receiver are functioning, but the screen shows a “No Connection” message.

Potential Cause: The connection between the HD Camera and Recording Unit has been interrupted, or the HD Camera has been damaged.

Solution: Power off the Recording Unit, reconnect the HD Camera, and turn on the Recording Unit.

10.2 Interrupted Wireless Connection

Symptom: The status indicator lights on the HD Video Receiver are not functioning.

Potential Cause: The wireless connection between the HD Camera and HD Video Receiver has been interrupted, or the wireless module has been damaged.

Solution: Power cycle the Recording Unit and restart the HD Video Receiver.

10.3 File Save Failure

Symptom: The screen shows a “Please insert SD Card” message.

Potential Cause: The SD card is not inserted or is damaged.

Solution: Power off the Recording Unit, reinsert the SD card. If problem persists the SD card might be damaged; insert a new SD card.

10.4 Image Blur

Symptom: The display image is blurred or obscured.

Potential Cause: The HD Camera has been turned to a non-ideal focal length or has come loose from its intended mounting location.

Solution: Reconnect the HD Camera to the desired mounting location and/or adjust the focus of the device by turning the front ring.

10.5 Artifacts in Image

Symptom: The display shows spots or aberrations.

Potential Cause: There is dust or smudges on the HD Camera lens.

Solution: Wipe the HD Camera lens with a lens tissue or cloth

11. END OF PRODUCT LIFE

This product contains electrical waste or electronic equipment. It must not be disposed of as unsorted municipal waste and must be collected separately in accordance with applicable national or institutional related policies relating to obsolete electronic equipment. The Wireless LED Headlight Camera System must be disposed of according to local laws and hospital practices.

CAUTION: Sensitive information may be stored on the internal micro SDXC card on the device. Please ensure all data is removed from the device before disposal.

WARNING: The Wireless LED Headlight Camera System is packed with lithium-ion battery cells. These batteries must be disposed of according to local regulations.

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